making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU

- ZH
- JA
- KO
- AR

SILENT EC2/TC2

Quick Start Guide

Made in Germany

21-6836 16072019

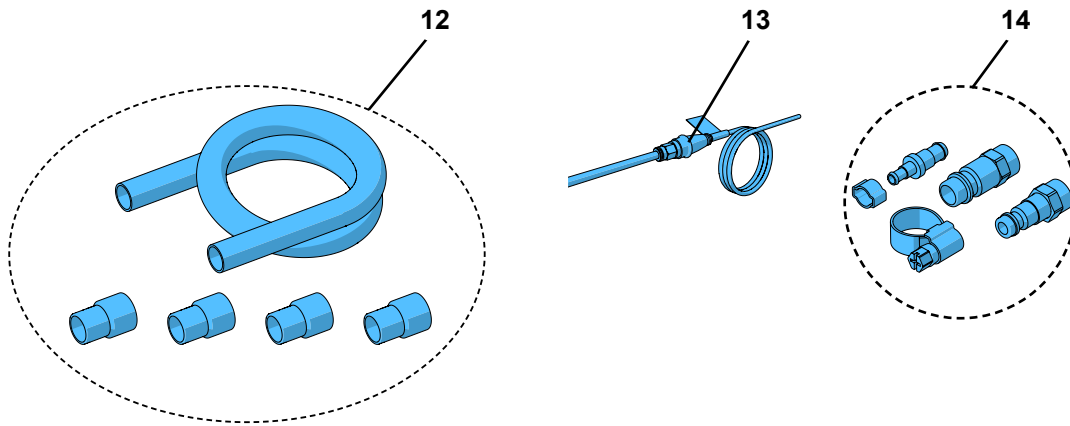
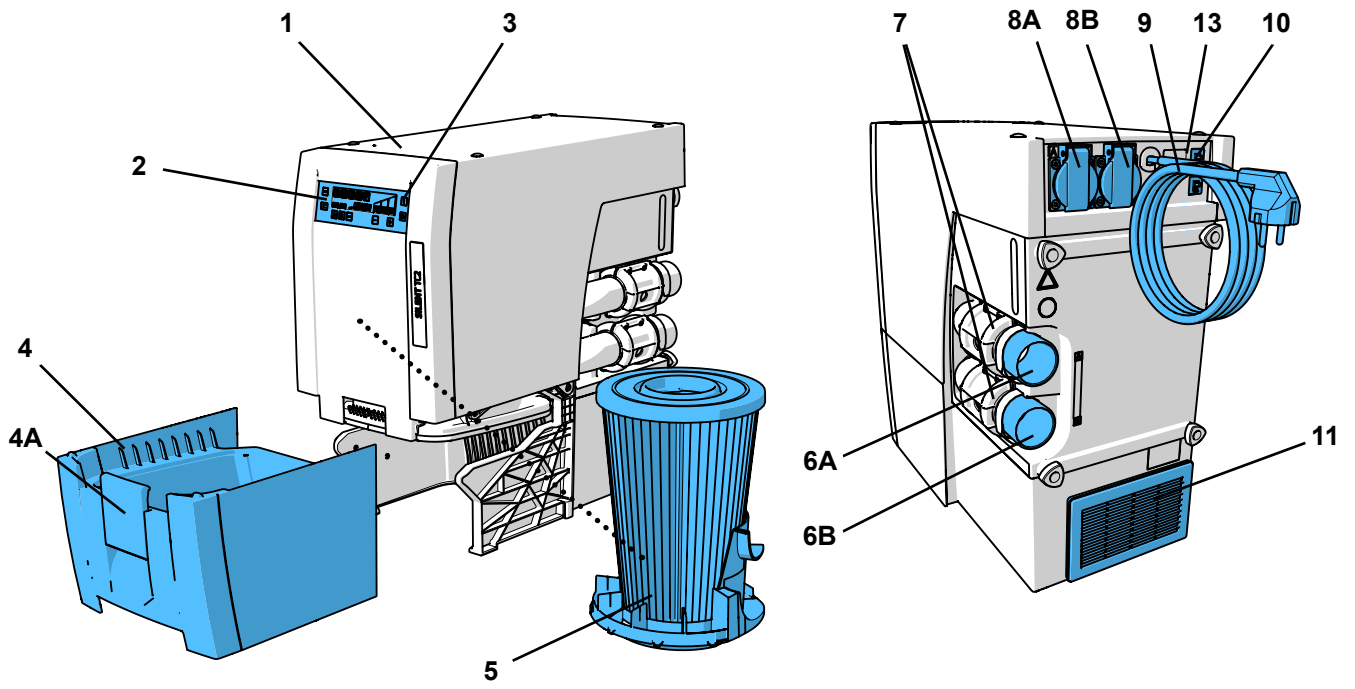
Fig. 1



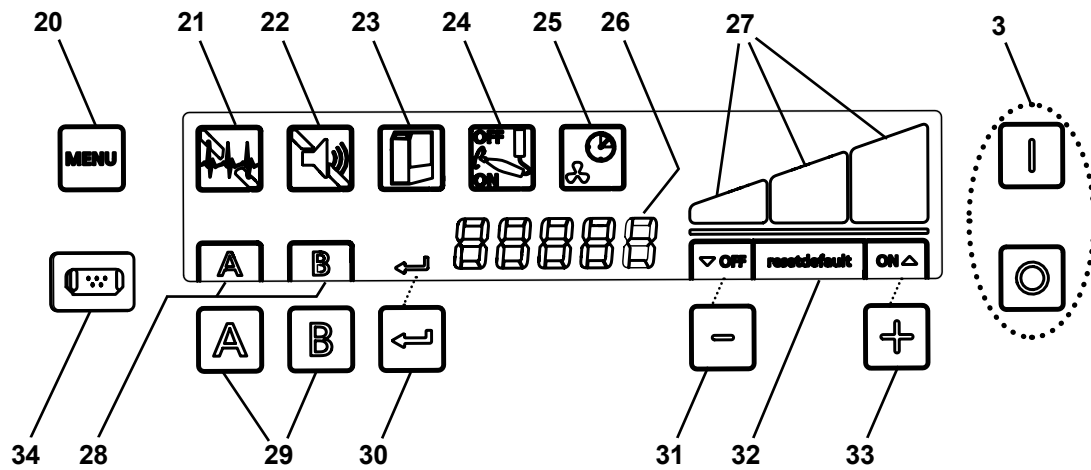
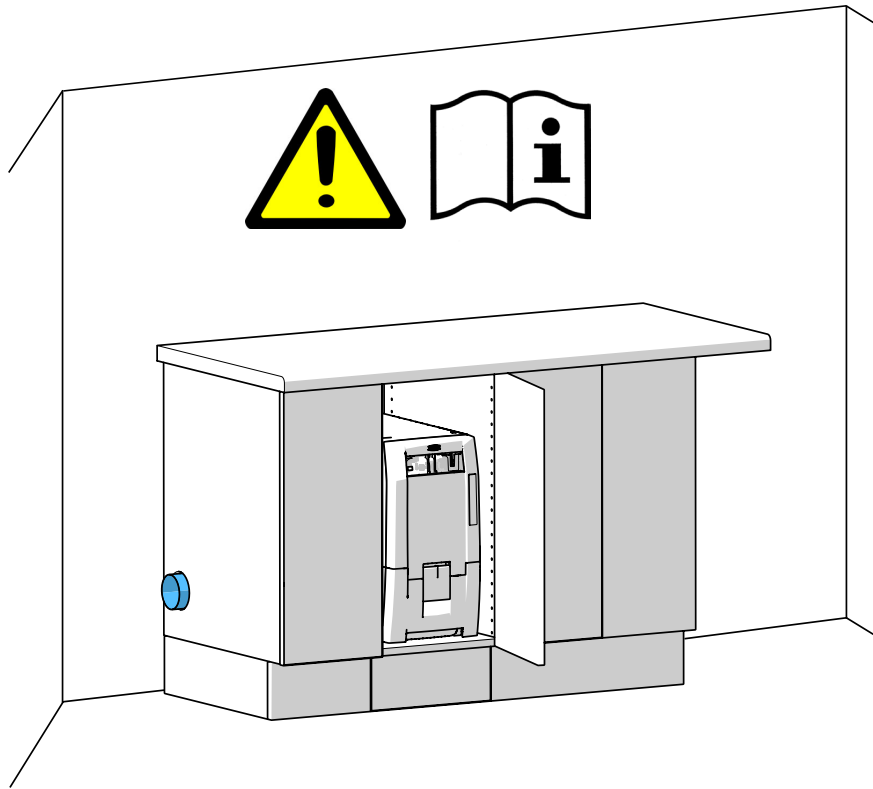
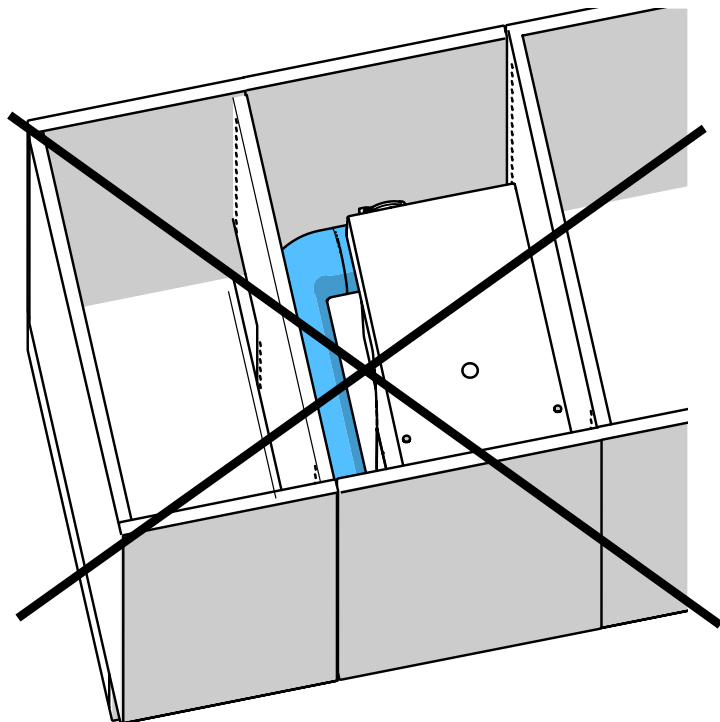
Fig. 2

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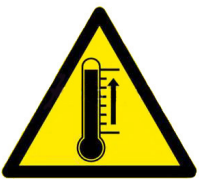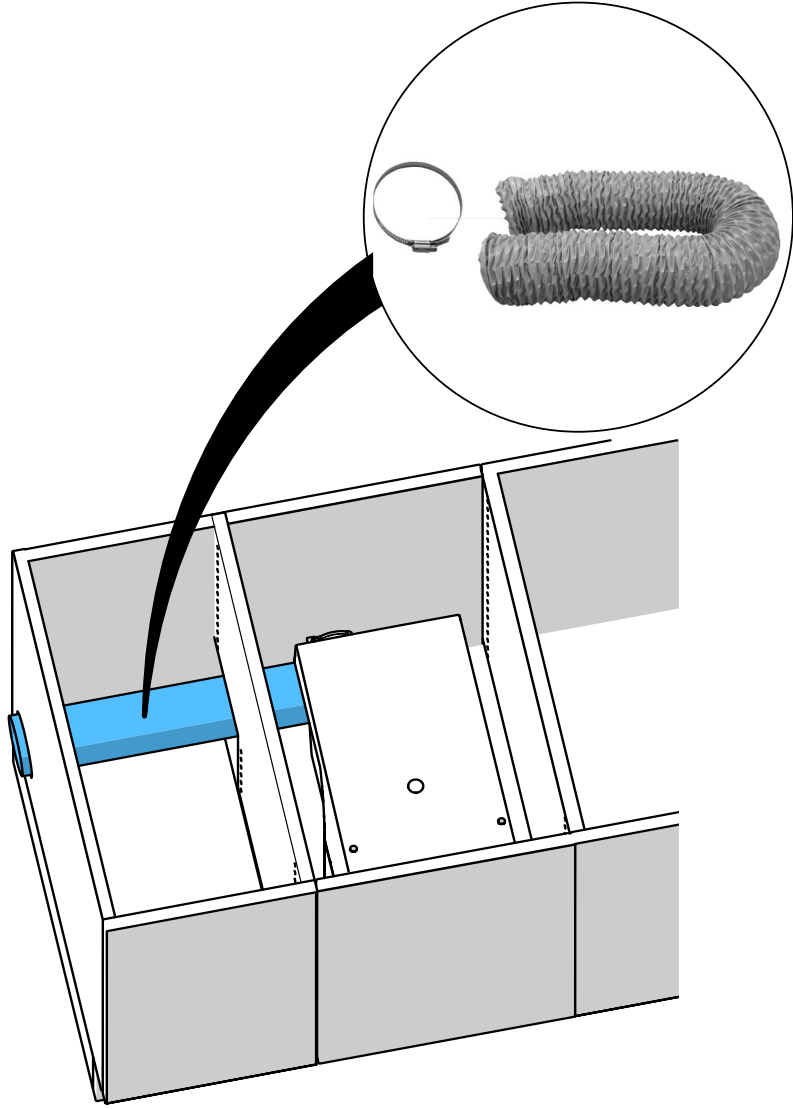Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

Detailed information on the product can be found in the operating instructions which are available for download on our website at www.renfert.com/p915.

Enter the following item number: EC2: 29370000, TC2: 29360000.

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Other symbols are explained as they occur.

Safety

Intended Use

This device is designed to extract dry, non-explosive dust.

The unit is intended solely for use in a commercial dental laboratory and dental practice.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

Improper Use

The equipment is not suitable for use with dental CAM units!

For information concerning SILENT extraction units for CAM machines, please refer to www.renfert.com.

Fire-promoting, easily flammable, red-hot, burning or explosive materials must not be suctioned into the device.

It is not permitted to suction liquids. This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device.

The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
After removing the Dust Drawer, you can find the identification plate below, inside the device on the left side.

- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Information

- ▶ The mains socket on the device is only designed for purposes specified in the operating instructions. Connecting other devices may cause material damage.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Always refer to the relevant safety data sheets, when extracting hazardous materials.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.
- ▶ Dispose of extracted material according to local statutory regulations.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Components Groups and Functional Elements

See Fig. 1 (on inside cover)

- | | | | |
|----|------------------------|----|---|
| 1 | SILENT EC2 / TC2 | 8A | Device socket A |
| 2 | Control panel | 8B | Device socket B |
| 3 | On / Off switch | 9 | Mains cable |
| 4 | Dust drawer | 10 | Protective device switch (2 x) |
| 4A | Clamp fastener | 11 | Exhaust air filter / Exhaust air outlet |
| 5 | Fine filters | 12 | Suction tube and end bushings |
| 6A | Suction port channel A | 13 | Compressed air connection tube
(incl. compressed air filter) |
| 6B | Suction port channel B | 14 | Connection set |
| 7 | Velcro® strip | | |

See Fig. 2 (on inside cover)

- | | | | |
|----|---------------------------------|----|---|
| 3 | On / Off switch | 27 | Display of the suction levels |
| 20 | Menu key | 28 | Suction channel indicator |
| 21 | Symbol self-diagnosis | 29 | Suction channel keys |
| 22 | Symbol acoustic signal | 30 | Enter key, save input / display enter key |
| 23 | Symbol dust drawer full | 31 | [-] key / display function of the [-] key |
| 24 | Setting the automatic switch-on | 32 | Display Factory setting |
| 25 | Set the after-run time | 33 | [+] key / display function of the [+] key |
| 26 | 7-segment display | 34 | Cover (do not remove, service interface) |

Initial Operation

Setting-up

The suction unit is a standing device which must not be operated in a horizontal position.

! **If the suction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the measures shown in Figs. A, B, C at the start of this manual.**

A: External ventilation duct (see accessories at www.renfert.com).

B: Observe the opening in the cabinet, specified measurements, minimum and maximum distances.

C: Remove the back side of the cabinet, the distance from the back side of the cabinet to the wall min. 50 mm.

Connect

⇒ Connect to the compressed air supply system, using a connection kit if necessary (14).

! **Compressed air should be clean and dry (without condensed water). Wet compressed air can damage the unit.**

Warning risk of injury!

When shortening the suction tube, please be sure to cut the integrated wire as straight as possible.

Long suction tubes, tight bends and kinks greatly reduce the suction power at the suction point.

⇒ Shorten the suction tubes to the required length for both suction points.

⇒ Screw the end fittings onto the suction tubes (left turn thread).

⇒ Place the suction tube onto the suction port (6A / 6B).

Observe the allocation of the device sockets (A, B) – suction channel (A, B).

⇒ Connect the suction tube to the correct suction point.

If the diameter is not suitable, use an adapter (see accessories), to avoid loss of suction.

⇒ Connect the electrical equipment to the device sockets (8A / 8B) on the back.

Observe the allocation of the device sockets (A, B) – suction channel (A, B).

When connecting the electrical equipment to the suction unit, ensure that the sum of the electrical power from the connected electrical appliances does not exceed the maximum permissible total power (see “Tech. Data” at the end of this document).

⇒ Switch the device off at the on / off switch (3).

⇒ Unroll the mains cable (9) and insert the plug into the building's electrical socket.

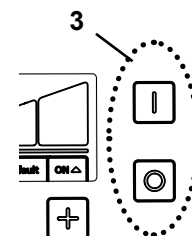
Operation

Switching On

The suction unit is switched on and off via the on / off switch (3).

After activation:

- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Stand-by

In the stand-by mode the display is dark.

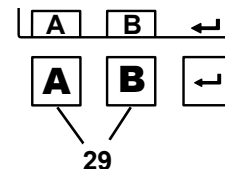
Leave stand-by mode:

- Press any button.
- Switch on any electrical appliance connected to the extractor.

Select vacuum channel

Start / stop suction turbine and open / close corresponding pinch valve valve:

- Automatically:
The electrical appliance connected to the device socket (8A / 8B) is activated / deactivated.
- Manually:
By pressing a suction hose button (29).

If the suction to a connected electrical appliance (e.g. a handpiece) does not react as required (e.g. does not activate, even though a connected device is in operation) then the switch-on threshold for automatic operation must be adjusted.

Suction Power

⇒ Press the [-] key (31): Reduce suction

⇒ Press the [+] key (33): Increase suction

Setting the Parameter

On delivery, there are parameters set, which in most cases, make it possible to work with the suction unit without problems. Only in individual cases, if this is not possible, the parameters can be changed.

The setting of various parameters and the execution of a self-diagnosis are carried out in the programming mode.

Information on how to set the parameters and perform a self-diagnosis can be found in the operating instructions in the Internet or in Figs. D and E at the start of this manual.

The display indicates which parameters are set:

Perform self-diagnosis

Acoustic signal (buzzer) on / off

Time interval for full display (full) in the dust drawer

Setting the automatic switch-on (calibration)

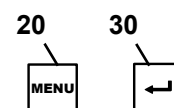
Set after-run (time) for the suction turbine

To set the different parameters, press the Menu key 20.


The settings are confirmed and saved with the Enter key 30.

Successful saving is acknowledged with an acoustic signal.

If a change is not accepted, pressing the Menu key (20) will abort the programming.

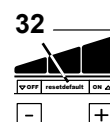If the word "Default" (32) appears in the display, you can reset the value to the factory setting by pressing the [-] and [+] keys at the same time.

Self-diagnosis (see Fig. D at the start of this manual)

Start the self-diagnosis manually. This is performed once.

Acoustic signal (see Fig. D at the start of this manual)

When switched on, various settings will be confirmed with an acoustic signal.

Time Interval for “Full” Display Dust Drawer (see Fig. D at the start of this manual)

After reaching a set time interval, a request appears to empty the dust drawer.

You can select between 5 times.

Time interval / hrs	Value in display (24)
2	1
5	2
10	3
50	4
100	5

Factory setting

i *With devices which generate large amounts of dust, (e.g. sandblasting units) the time interval “empty dust drawer” should be set to 5 hours, if necessary, to 10 hours.*

Automatic Switch-on for Automatic Operation (see Fig. E at the start of this manual)

The switch-on threshold, which activates and deactivates the suction function when a connected electrical device is operated, can be set.

When setting the switch-on threshold for handpieces, the control unit must be switched on for the first measurement (OFF) without operating the handpiece.

In the second measurement (ON), handpieces should be operated at the speed at which the suction is to be switched on.

i *Devices with a stand-by mode should be switched on approx. 5 minutes before setting to avoid incorrect measurements.*

i *If there is an error with the switch-on function, the display “Cal” will blink and an acoustic signal is emitted twice. For handpieces, select a higher speed if necessary. See also “Eliminate faults” in the complete operating instructions, which you can download from our website at www.renfert.com/p915.*

After-run Time of the Suction Turbine (see Fig. E at the start of this manual)

After-run time = The period of time between switching a connected device off and the closing of the corresponding suction channel (= pinch valve).

A distinction is made between three after-run times, which can be set independently of one another (factory setting: 3 seconds):

- After-run time suction channel A → only suction channel A is open:
Time, until the pinch valve of suction channel A closes and the suction turbine switches off.
- After-run time suction channel B → only suction channel B is open:
Time, until the pinch valve of suction channel B closes and the suction turbine switches off.
- After-run time suction channel A and B → both suction channels, A AND B, are open:
Time, until the pinch valve of a suction channel closes and the suction performance for the other suction channel is adjusted.

i *In order to prevent the suction from being switched on / off too often, or the suction performance to be adjusted too frequently in the case of short interruptions (for example when working with the handpiece), a higher time value should be selected.*

The setting after-run time can be exited at any time before or after saving the entered value via the Menu key (20).

Cleaning / Maintenance

The device does not contain any parts that require maintenance.
Opening the device other than described in the instructions is not permitted!

Cleaning

To clean the device wipe only with a damp cloth.
Do not use and solvent-based or abrasive cleaning agents.

Empty Dust Drawer

After reaching the set time interval, the dust drawer is requested to be emptied:

- ◆ An acoustic signal sounds 3 x.
- ◆ The symbol “dust drawer full” is shown.
- ◆ In the display “FULL” is shown.

Before removing the dust drawer, a filter cleaning function should be performed:

⇒ Press Enter key (30) for 2 seconds.

- ◆ The filter cleaning function is performed (duration approx. 8 seconds).

After the filter cleaning has finished:

- ⇒ Unlock the dust drawer with the clamp fastener (4A).
- ⇒ Pull the dust drawer (4) forward and empty.
- ⇒ Replace the dust drawer again until it locks into place.
- ⇒ Close the clamp fastener (4A).
- ⇒ Press the Enter key (30) (counter for recording the time interval is reset).
 - ◆ Acoustic signal acknowledges the information.
 - ◆ Dust drawer display (23) is deleted.

If the dust drawer is not emptied, the “Dust drawer full” and “FULL” symbols are still shown in the display. After switching the device off and on again, a 3 x acoustic signal is used to indicate the emptying of the dust drawer.

To set the time interval, see Fig. D at the start of this manual, or find information in the instructions for use in the Internet.

Change fine filter

If the filter cleaning does is not sufficient (suction performance does not improve), then the fine filter needs to be changed.

This is detected by the suction system and indicated by “FILt” in the display.

When installing the fine filter, pay attention to the correct fit, otherwise there will be leakage. Please refer to the assembly guide at the end of this manual, which is also included with the new fine filter.

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: EC2: 29370000, TC2: 29360000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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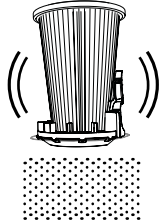
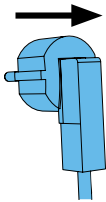
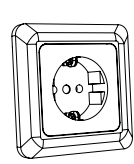
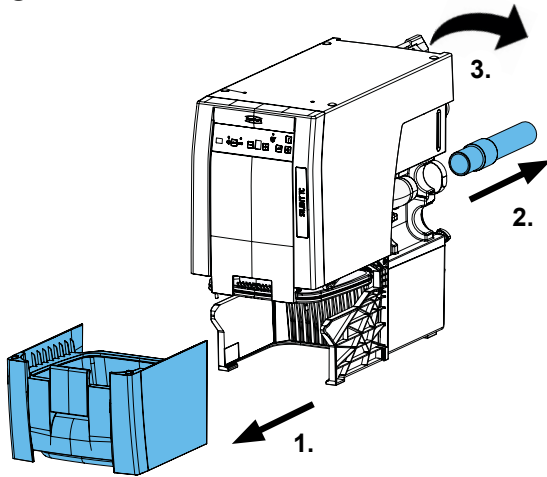
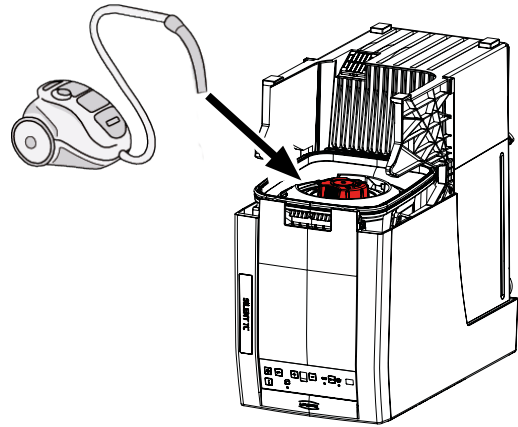
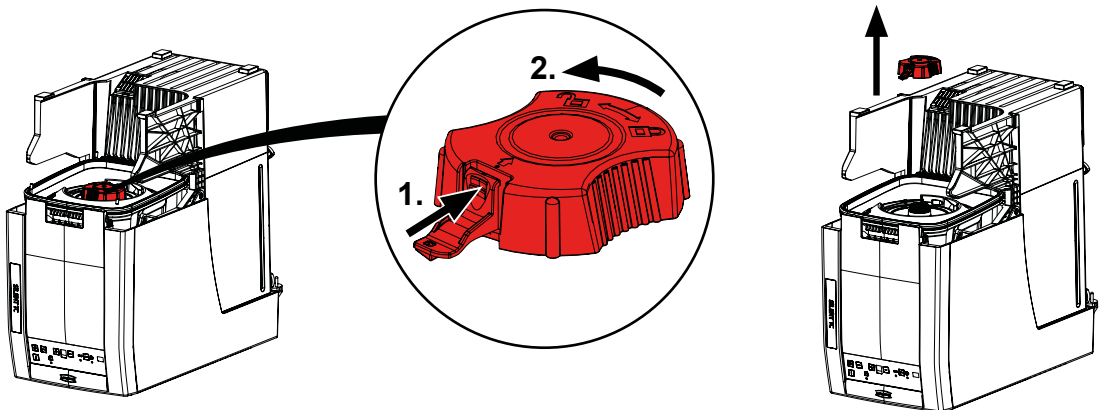
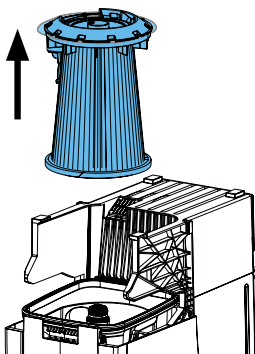
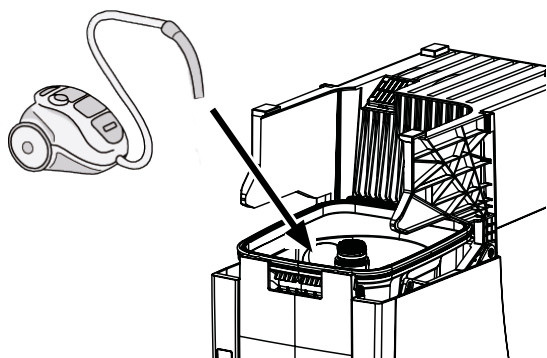
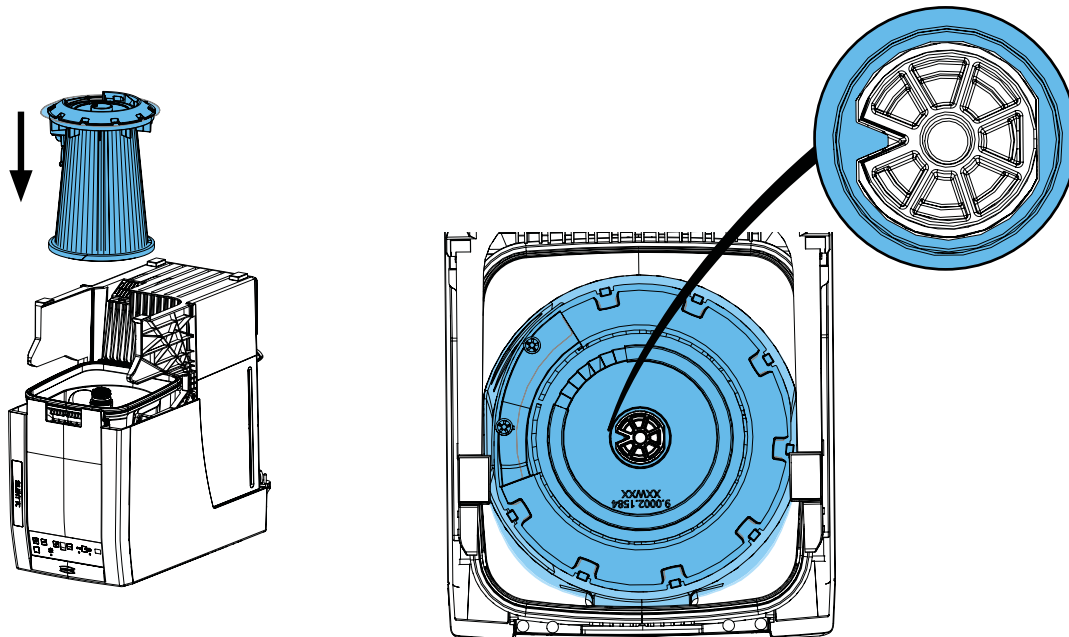
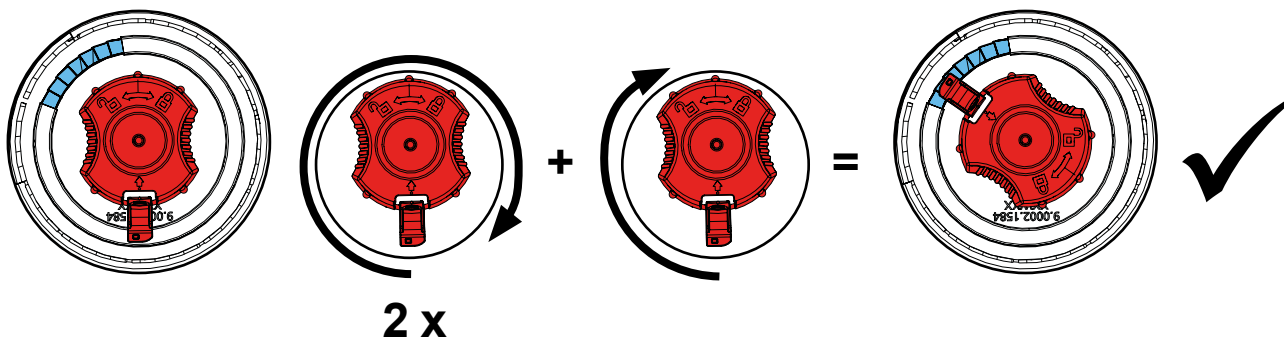
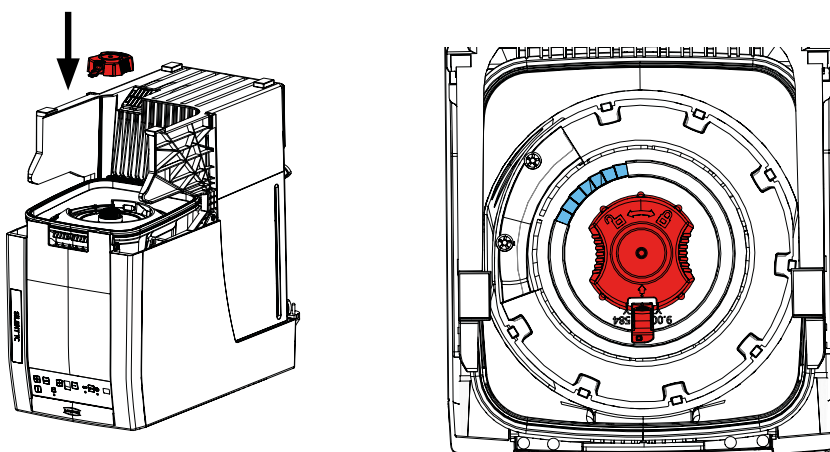
9.

Technical Data

	SILENT TC2	SILENT EC2		
	2936 0000	2937 0000	2937 1000	2937 2000
Working Voltage:	230 V	230 V	120 V	100 V
Permissible mains voltage:	220 - 240 V	220 - 240 V	120 V	100 V
Mains frequency:	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Suction turbine power: *)	850 VA	1610 VA	1440 VA	1150 VA
max. connecting power: *)	2000 W	1840 W	430 W	350 W
Total connected power: *)	2850 W	3450 W	1800 W	1500 W
Mains input fuse:	2 x 15 A (T)			
max. connection pressure:	8 bar [116 psi]			
min. connection pressure:	4.5 bar [65 psi]			
LpA **) (at max. volume flow):	55 dB(A)			
Number of suction hoses:	2			
Ø Suction fittings: - internal: - external:	45 mm [1.77 inch] 50 mm [1.97 inch]			
max. Volume flow:	3980 l/min [2.34 ft ³ /s] ***)			
Vacuum pressure, max.:	281 hPa [4.10 psi]	257 hPa [3.73 psi]	262 hPa [3.80 psi]	255 hPa [3.70 psi]
Fine particle filter: - Filter surface area, approx.: - Filterquality:	0.9 m ² [1390 sq inch] Class M according to EN 60335-2-69			
Fill level dust drawer, approx.:	7 l [1.85 US gal]			
Dimensions (Width x Height x Depth):	285 x 535 x 540 mm [11.2 x 21.1 x 21.2 inch]			
Weight (empty), approx.:	23 kg [50.7 lbs]	18.5 kg [40.8 lbs]		

*) At nominal voltage

**) Sound pressure levels according to EN ISO 11202

***) Free blowing turbine

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

www.renfert.com

Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70
www.renfert.com • info@renfert.com

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87
www.renfert.com • info@renfertusa.com
USA: Free call 800 336 7422