making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU
  
- ZH
- JA
- KO
- AR

# SILENT TC

Quick Start Guide

Made in Germany

21-6832 12072019

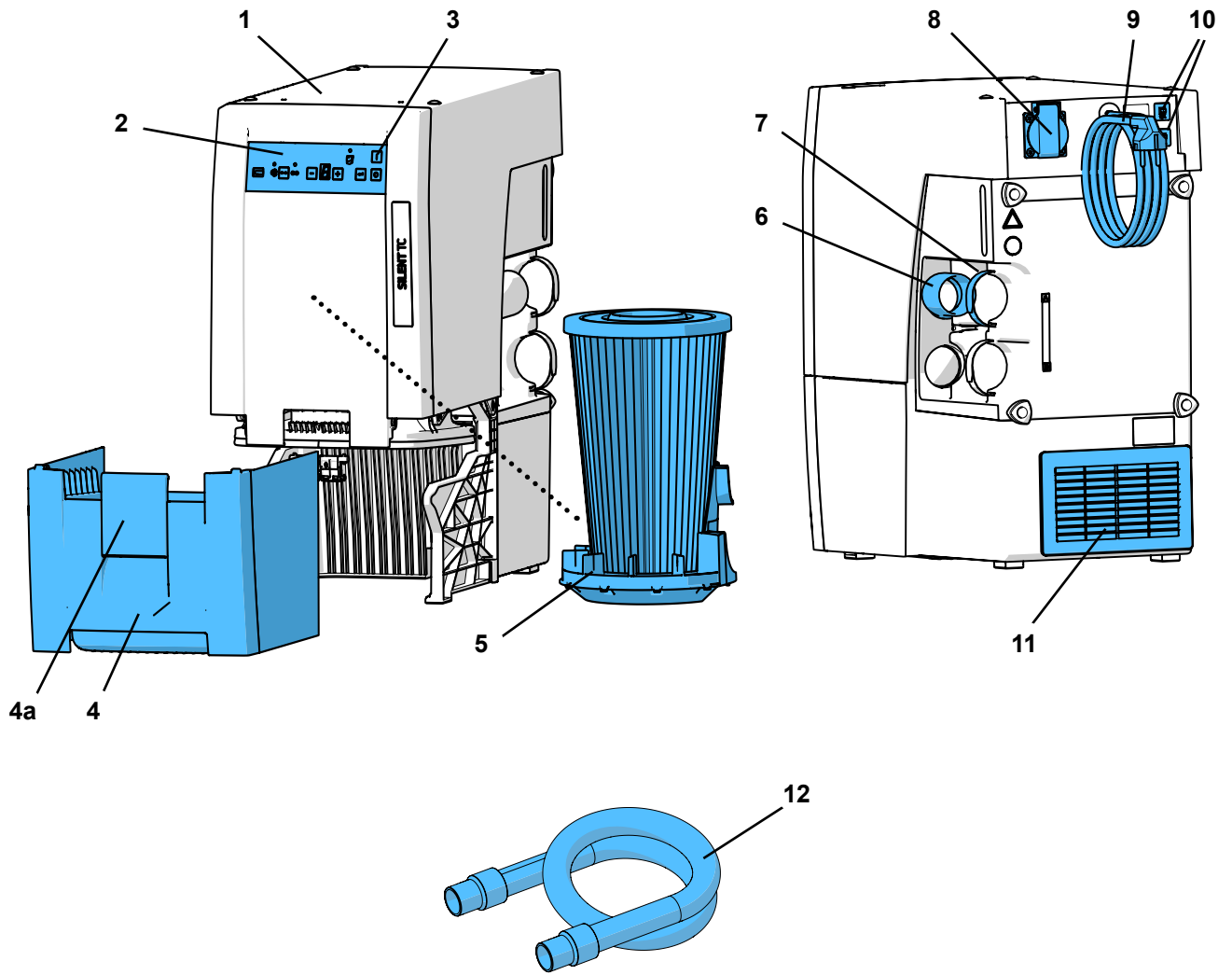
Fig. 1


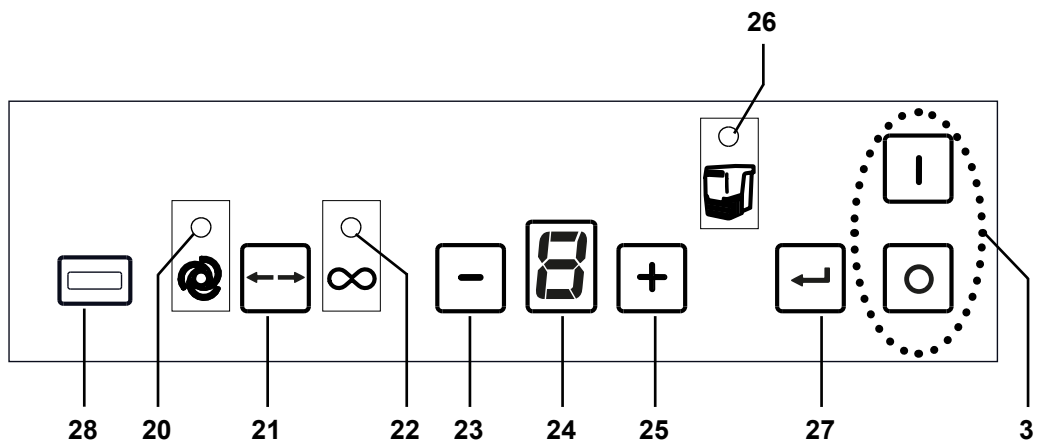
Fig. 2

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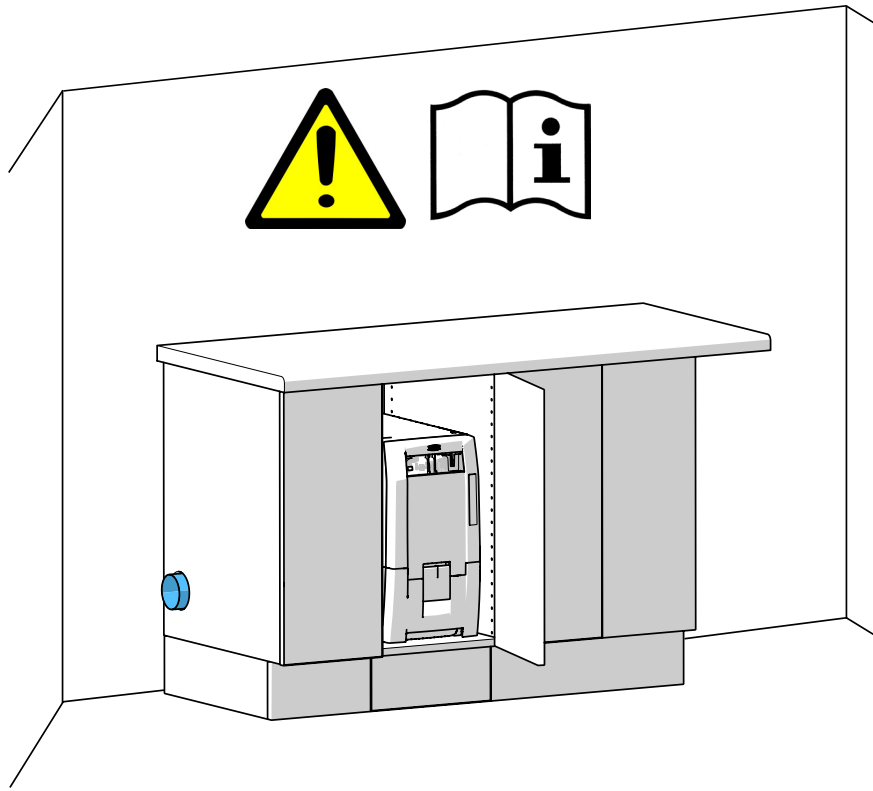# Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website at [www.renfert.com/p915](http://www.renfert.com/p915).

Enter the following item number: 29350000.

## Symbols used

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

### Danger

**Immediate risk of injury. Consult accompanying documents!**

### Electrical current

**This indicates a risk of injury due to electrical current.**

### Caution

**Risk of damage to the unit if the instruction is not followed.**

### Notice

**A general notice that improves and eases use.**

**The appliance complies with applicable EU directives.**

**Within the EU, this device is subject to the provisions of the directive 2002/96/EC (WEEE directive).**

Other symbols are explained as they occur.

## Safety

### Intended Use

This device is designed to extract dry, non-explosive dust.

The unit is intended solely for use in a commercial dental laboratory and dental practice.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

### Improper Use

**The equipment is not suitable for use with dental CAM units!**

**For information concerning SILENT extraction units for CAM machines, please refer to [www.renfert.com](http://www.renfert.com).**

Fire-promoting, easily flammable, red-hot, burning or explosive materials must not be suctioned into the device. It is not permitted to suction liquids. This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device. The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

### Hazard and Warning Information

#### General Information

- ▶ If the device is not operated according to the enclosed documents, the provided protection is no longer guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.  
After removing the Dust Drawer, you can find the identification plate below, inside the device on the left side.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.

- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Regularly check connecting cables (such as power cables), hoses and housing (for example, the control panel) for damage (for example, kinks, tears, porosity) or deterioration due to aging. Do not operate units with damaged connection cables, hoses or housing parts or other defects!
- ▶ Defective devices must be put out of service immediately. Disconnect from mains and secure against being switched on again. Send the device in for repair!
- ▶ Only operate the device under supervision.
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### Specific Information

- ▶ The mains socket on the device is only designed for purposes specified in the operating instructions. Connecting other devices may cause material damage.
- ▶ Switch off the electrical device before connecting an electrical appliance to the appliance outlet.
- ▶ Read the operating instructions for the electrical appliance which is to be connected and observe the stipulated safety instructions.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Always refer to the relevant safety data sheets, when extracting hazardous materials.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.
- ▶ Dispose of extracted material according to local statutory regulations.

### Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

## Product Description

### Component Groups and Functional Elements

See Fig. 1 (on inside cover)

- |                   |  |
|-------------------|--|
| 1 SILENT TC       | 7 Velcro® strip                            |
| 2 Control panel   | 8 Device socket                            |
| 3 On / off switch | 9 Mains cable                              |
| 4 Dust drawer     | 10 Protective device switch (2 x)          |
| 4a Clamp fastener | 11 Exhaust air filter / Exhaust air outlet |
| 5 Fine filter     | 12 Suction hose                            |
| 6 Suction port    |  |

See Fig. 2 (on inside cover)

- |  |   |
|--|---|
| 3 On / off switch  | 24 Display                                  |
| 20 Display Automatic operation                           | 25 [+] -key (increase value)                |
| 21 Start operation key, Automatic / continuous operation | 26 Display empty dust drawer                |
| 22 Display continuous operation                          | 27 Enter key, save input                    |
| 23 [-] key (reduce value)                                | 28 Cover (do not remove, service interface) |

# Initial Operation

## Setting-up

The suction unit is a standing device which must not be operated in a horizontal position.

**!** If the suction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the measures shown in Figs A, B, C at the start of this manual.

A: External exhaust air duct (see accessories at [www.renfert.com](http://www.renfert.com)).

B: Opening in cabinet rear panel, specified dimensions, observe the minimum and maximum measurements.

C: Remove the back panel of the cabinet, distance from the back of the cabinet to the wall min. 50 mm.

## Connect

**!** **Warning risk of injury!**  
When shortening the suction tube, please be sure to cut the integrated wire as straight as possible.

**i** Long suction tubes, tight bends and kinks greatly reduce the suction power at the suction point.

⇒ Shorten the suction tube if possible.

⇒ Open the adhesive strap (7).

⇒ Plug the suction tube end piece (12) into the suction socket (6).

⇒ Fix the suction tube in place with the adhesive strap.

⇒ Connect the suction tube to the correct suction point.

**i** If the diameter is not suitable, use an adapter (see accessories), to avoid loss of suction.

⇒ Connect the electronic device to the appliance socket (8) on the back.

**!** When connecting electrical appliances to the suction unit, ensure that the maximum permissible power for connected devices is not exceeded (see "Technical data" at the end of this document).

⇒ Switch the device off at the on / off switch (3).

⇒ Unroll the power cord (9) and insert the power plug into the building wall socket.

# Operation

## Switching On

The suction unit is switched on and off via the on / off switch (3).

After activation:

- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Afterwards, the suction unit is in the last set mode.

## Select operating mode: Automatic operation / continuous operation

- Automatic mode (20):

The suction is dependent on the electrical device connected to the appliance outlet (8).

- Permanent operation (22):

Suction running continuously.

⇒ Press operating mode key (21).

- ◆ Changing the operating mode.

**i** If the suction to a connected electrical appliance (e.g. a handpiece) does not react as required (e.g. does not activate, even though a connected device is in operation) then the switch-on threshold for automatic operation must be adjusted.

## Suction Power

The suction power for the suction unit can be set in 4 levels.

⇒ [-] key (23): Reduce suction performance

⇒ [+] key (25): increase suction performance

## Setting the Parameter

On delivery, there are parameters set, which in most cases, make it possible to work with the suction unit without problems. Only in individual cases, if this is not possible, the parameters can be changed.

The setting of various parameters and the execution of a self-diagnosis are carried out in the programming mode.

**Please refer to the operating instructions on the Internet or Figs. D and E at the beginning of this document to find out how to set the parameters and run self-diagnosis.**

The display (24) shows which parameter are set:

	„d“	Perform self-diagnosis
	„b“	Acoustic signal on / off (decimal point on / off)
	„A“	Time Interval for “Full” Display Dust Drawer
	„c“	Setting the automatic switch-on (calibration)
	„t“	Set after-run (time) for the suction turbine

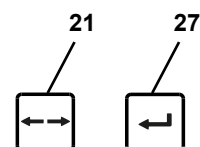
In order to set the different parameters, the programming mode must be started and the parameter selected.

To do so, press the mode key (21) and enter key (27), see Figs. D and E at the beginning of this document.

Change values with the [-] key and [+] key.

Confirm and save settings with the Enter key (27). Successful saving is acknowledged with an acoustic signal.

If a change is not to be applied, press the mode key (21) again to cancel.

### Self Diagnosis (see Fig. D at the beginning of this document)

Start the self diagnosis manually. This is carried out once.

### Acoustic signal (see Fig. D at the beginning of this document)

When on, various inputs are confirmed with an acoustic signal. This beep can be switched on and off.

### Time interval for full display of the dust drawer (see Fig. D at the beginning of this document)

After reaching a set time interval, a request appears to empty the dust drawer.

It is possible to select one of 5 different times.

Time interval / hrs	Value in display (24, Fig. 2)	
2	1	
5	2	
10	3	
50	4	Factory setting
100	5	

**In the case of heavy dust generators (e.g. blasting equipment), the time interval "emptying the dust drawer" should be set to 5 hours, if necessary to 10 hours.**

### Automatic switch-on for automatic operation (see Fig. E at the beginning of this document)

The switch-on threshold, which activates and deactivates the suction function when a connected electrical device is operated, can be set.

When setting the switch-on threshold for handpieces, the control unit must be switched on for the first measurement (stand-by) without operating the handpiece

For the second measurement (On), the handpiece should be operated at the speed at which the suction is to be switched on.

**i** For devices with standby mode, these should be switched on approx. 5 minutes before setting to avoid incorrect measurements.

**i** If there is an error with the switch-on function, the display “C” will blink and an acoustic signal is emitted twice. Set the changes again. If necessary, select a higher speed for handpieces. For detailed information on "Troubleshooting", see the instructions for use which you can download from our website at [www.renfert.com/p915](http://www.renfert.com/p915).

### After-run time of the suction turbine (see Fig. E at the beginning of this document)

After-run time = the time between switching off a connected consumer and turning off the suction turbine. There are 6 different times to choose from.

After-run time / sec.	Value in display (24, Fig. 2)
0	1
3	2
5	3
10	4
20	5
30	6

Factory setting

**i** In order to prevent the suction from being switched on / off too often in the case of short interruptions (for example when working with the handpiece), a higher time value can be selected.

## Cleaning / Maintenance

The device does not contain any internal parts that require maintenance. Opening the device other than described in the instructions is not permitted!

### Cleaning

To clean the device wipe only with a damp cloth.  
Do not use and solvent-based or abrasive cleaning agents.

### Empty Dust Drawer

After reaching a set time interval, a request appears to empty the dust drawer:

- ◆ An acoustic signal sounds 3 x.
- ◆ Display dust drawer (26) lights up.
- ◆ The display (24) shows a container filling up.

Before removing the dust drawer, a filter cleaning function should be performed:

⇒ Press Enter key (27) for 2 seconds.

- ◆ The filter is cleaned (duration approx. 8 sec).
- ◆ The counter for recording the time interval is reset.

After the filter cleaning has finished:

- ⇒ Unlock the dust drawer with the clamp fastener (4a).
- ⇒ Pull the dust drawer (4) forward and empty.
- ⇒ Replace the dust drawer again until it locks into place.
- ⇒ Close the clamp fastener (4a).

If the dust drawer has been emptied without prior filter cleaning, the counter must be reset separately to record the time interval.

⇒ Press the Enter key (27) (counter for recording the time interval is reset).

- ◆ Acoustic signal acknowledges the information.
- ◆ Dust drawer display (26) is deleted.

**i** If the dust drawer is not emptied, the dust drawer (26) display will still light up. After switching the device off and on again, a 3 x acoustic signal is used to indicate the emptying of the dust drawer.

To set the time interval, see Fig. D at the beginning of this document, or refer to the Instructions for Use on the Internet.

## Change Fine Filter

If the filter cleaning does not show sufficient effect (suction power has not improved), the fine filter must be changed.

This is detected by the suction and indicated by "A" in the display.

**If requested to change the fine filter or the suction performance does not improve substantially or only for a short period, the fine filter must be changed soon. Further operation can lead to damage to the device.**

**Perform filter cleaning before changing the fine filter.**

When installing the fine filter please ensure that it is positioned correctly, otherwise leakages may result. Please refer to the assembly instructions at the end of this document, which are also included with the new fine filter.

## Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 29350000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

## Disposal Information

**Within the European Union, units with this symbol should therefore not be disposed of in unsorted domestic waste.**

Please contact your local authorities for more information on proper disposal.

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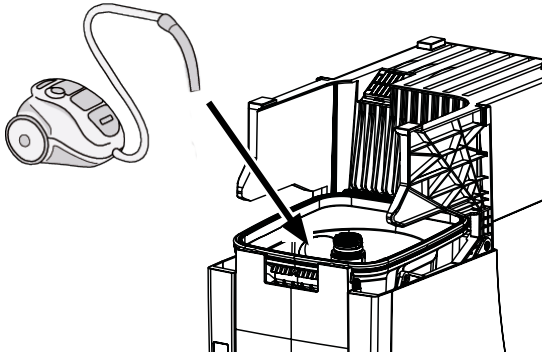
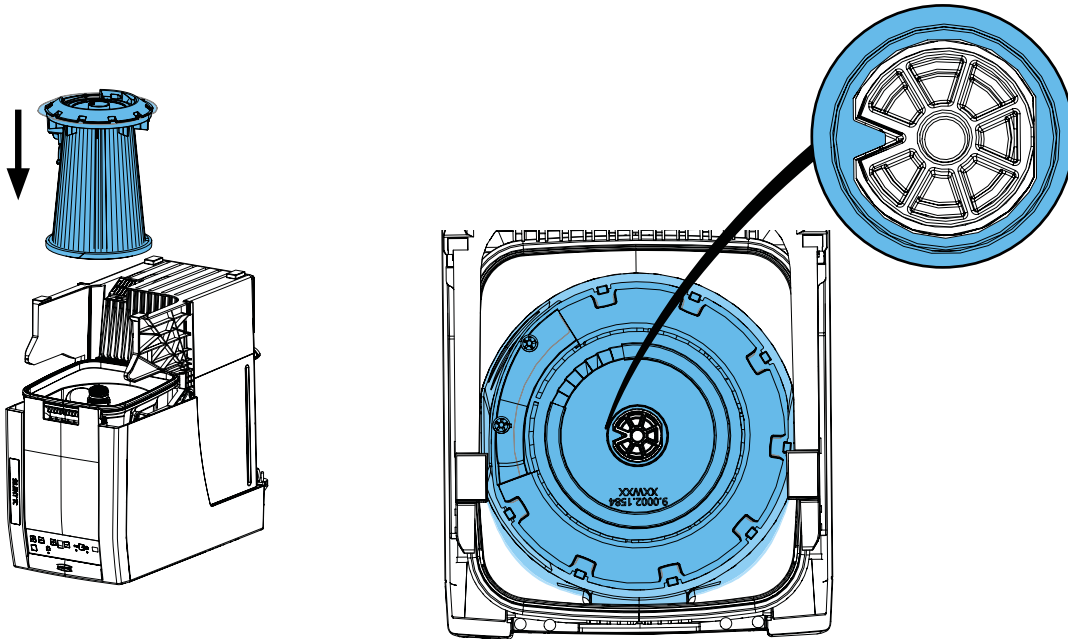
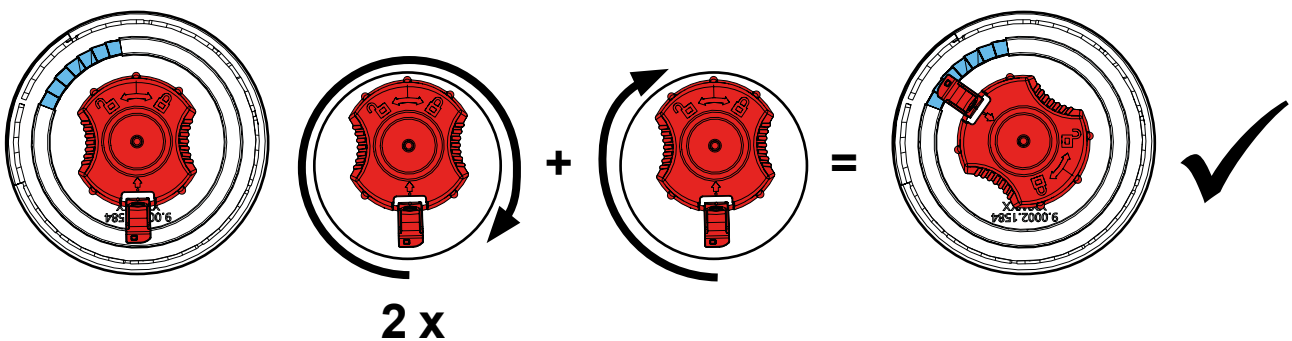
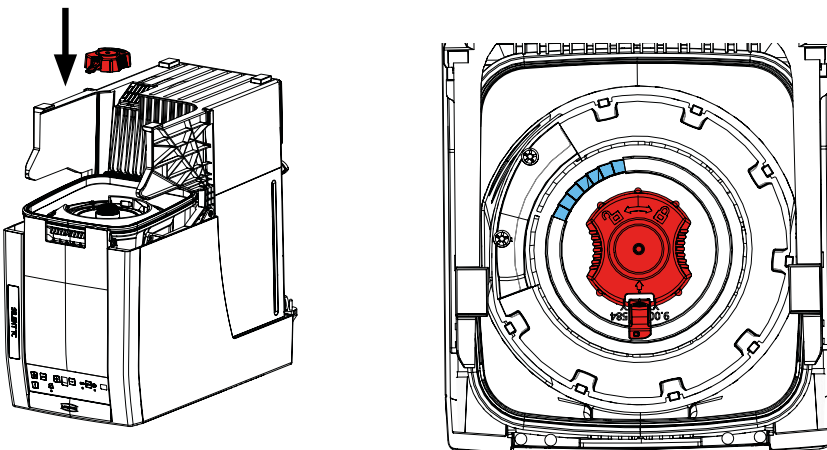
9.

# Technical Data

<b>Working Voltage:</b>	<b>230 V</b>
Permissible mains voltage:	220 - 240 V
Mains frequency:	50 / 60 Hz
Suction turbine power: *)	850 VA
max. connecting power: *)	2000 W
Total connected power: *)	2850 W
Mains input fuse:	2 x 15 A (T)
LpA **) (at max. volume flow):	54,3 dB(A)
Ø Suction fittings: - internal: - external:	45 mm [1.77 inch] 50 mm [1.97 inch]
max. Volume flow:	3980 l/min [2.34 ft <sup>3</sup> /s]
Vacuum pressure, max.:	281 hPa [4.1 psi]
Fine particle filter: - Filter surface area, approx.: - Filterquality:	0.9 m <sup>2</sup> [1390 sq inch] Class M according to EN 60335-2-69
Fill level dust drawer, approx.:	7 l [1.85 US gal]
Dimensions (Width x Height x Depth):	275 x 535 x 540 mm [10.8 x 21.1 x 21.3 inch]
Weight (empty), approx.:	22 kg [48.5 lbs]

\*) At nominal voltage

\*\*) Sound pressure levels according to EN ISO 11202

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422

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c/o Renfert GmbH